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GB Instruction Manual

THE BASIC PRINCIPLE OF ULTRASONIC CLEANING

The cleaner uses ultrasonic energy (40 kHz) in the form of sound waves to create millions of tiny microscopic bubbles in the solution that even works its way into holes and hidden cavities, loosening dirt on all surfaces that the solution touches. This action called cavitation, occurs thousands of times per second to gently yet thoroughly scrub contamination off the article being cleaned. When you lift the item out of the cleaner, it's microscopically clean.

NOTES BEFORE USE

1. Ensure the power supply is in rated range before power cable is connected. Refitting is strictly prohibited! Attention that the control panel will be eroded by organic solution, strong acid and strong alkali.
2. Ensure the earth wiring is well connected before starting.
3. Ensure the power switch should be placed in the "0" position before starting.
4. Do not operate if the tank is empty, or ultrasonic generator will be damaged. If heating is needed, the water level should not be less than 2/3.
6. Please close the lid to reduce noise and pay attention to water and steam in case of burning skin while open the lid.
7. Do not relocate the machine when fluid in the tank in case of overflow.
8. Please choose the proper sizes or types of baskets according to the object to be cleaned
9. Suggest using water-soluble liquid for benchtop ultrasonic cleaners. Strong acid or flammability cleaner is forbidden. Do not heat when using flammable liquid such as alcohol, gasoline, etc., heating is strictly prohibited, ultrasonic vibration for a long time cause temperature of liquid increase, caution fire!

10. Do not use the machine in severe environment: The place where temperature changes fiercely; The place where humidity is too high and is easy to produce dew; The place where vibration or impact is strong; The place where exists corrosive gas or dust; The place where water, oil or chemicals splash; The place where is filled with explosive and flammable gas.

Model	ZX-020S
Frequency (KHz)	40
Capacity (L)	3,2
Tank size (mm)	240 x 135 x 100
Unit size (mm)	265 x 165 x 220
Heating power (W)	100
Ultrasonic power (W)	120
Time (M)	0-30 min
Temperature (°C)	room Temp. ~80°C

OPERATION STEPS

Before Operating Your Cleaner

1. Add cleaning liquid

Add appropriate amount of cleaning solution to the cleaning tank according to the cleaning agent producer's instructions. Ensure that the liquid level is not lower than the inner tank height by 2/3 when the heating is turned on. Do not heat flammable liquids such as alcohol, gasoline, etc. It is forbidden to heat. Long-term vibration will cause liquid increase in temperature, beware of fire!

2. Power on

Plug the cleaner into a grounded outlet. Turn on the power switch at the tank back, then machine is power on.

Start ultrasonic cleaning

For the first Boot, time display show setting time at random, press touch time key + one time, "the setting time" on the display increase by 1 minute correspondingly, Touch time - Key to decrease time setting by 1 minute. Touch ON/OFF switch after set the time well, ultrasonic indicator turns red and machine starts working based on specified the parameters, meanwhile timer begin to count down and stop working until time shows "00:00", automatically.

Heat

The temperature is randomly displayed at the first boot, and the temperature can be set. Each time the temperature + key is pressed, the temperature increases by 1°C, and each time the temperature - key is pressed, the temperature decreases by 1°C. After setting the temperature, press the ON/OFF button and the machine starts to heat up. When the temperature reaches the set temperature, the heating indicator light goes out.

Power OFF

Press the ultrasonic "ON/OFF" button, the ultrasonic stops and the light is off. Press the heater "ON/OFF" button, the heating stops and the light is off. And then cut off the power, empty out the liquid, clean the tank and the unit with cloth for next use.

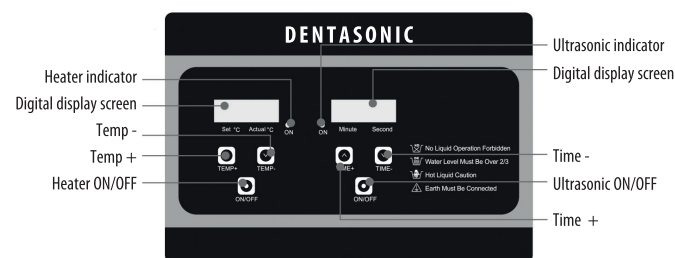
APPLICATION

The full range of standard ultrasonic cleaning machines are made of high-quality stainless steel plate, which is corrosion-resistant and has a long service life. It adopts high-quality ultrasonic transducers and advanced bonding technology, with high electro-acoustic conversion efficiency and strong ultrasonic power output. Equipped with automatic constant temperature heating device, heating range: room temperature ~ 80.

MAINTENANCE

Maintenance

Regularly use high-pressure wind blown away dust for electrical parts Clean the tank at least once a week

Troubleshooting

No	Problem	Possible causes	Solutions	Remarks
1	No Ultrasonic	A. Power supply not connect B. Fuse broken C. Cable short circuit D. Transducer short circuit E. PCB Board broken F. Other reasons	A. Check and plug power switch B. Check the fitted power supply and change the same specification fuse C. Connect fitted cable or replace a new D. Inquiry our after service engineer E. Check the broken part and change it F. Inquiry our after service engineer	
2	Time control failure	A. Timer failure B. other reasons	A. Replace timer or digital control panel B. Inquiry our after service engineer	
3	No heating	A. Heater power switch bad linkage B. Fuse burn out C. Heating pad burn out D. Digital display board out of service E. Incorrect temperature setting F. Other reasons	A. Check heating plug and connect well. check outlet line with multimeter if OK and resistance value is few hundred OHMs. B. Change the same specification fuse C. Replace the bad heating pad if open circuit D. Check the broken part and change it E. The set temperature must be 3°C higher than the actual temperature before heating can be turned on. Please reset the temperature. F. Inquiry our after service engineer	Suggestion 50-60°C
4	Temperature control failure	A. Thermostat loosen B. Thermostat tube broken C. Digital display out of service D. Other reasons	A. Fasten the thermostat header. B. Replace thermostat. C. Check the broken part and change it. D. Inquiry our after service engineer	
5	Not well cleaning	A. too high too low liquid surface B. too high too low temperature C. Not suitable cleaning liquid D. Other reasons	A. Adjust liquid into the best surface B. Adjust temperature into the most fitted C. Stop and switch off power supply, replace suitable liquid after the previous liquid cool down D. Inquiry our after service engineer	Suggestion 50-60°C
6	Electric leakage	A. The equipment ground wire is not connected well. B. The internal live line or neutral line of the equipment falls off	A. Check whether the internal ground wire and power cable grounding of the device are connected. B. Check if the internal live wire or neutral wire of the machine is fall off.	
7	Scrap	Not repairable or discarded	Disposal in accordance with EC directives - This appliance must not be disposed of with household waste! - Disposal exclusively by certified specialist companies	
8	Other problems		Inquiry our after service engineer	

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