AMALGAMPISTOLE

Stand:120321

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GB Instructions for use

The amalgam-gun is a medical device designed for amalgam applications, it is manufactured with self-lubricating material.

Push the tip of the syringe pushing amalgam on the compound to be used in order to fill the hollow cylinder of the same.

Insert the amalgam into the cavity and press the amalgam pusher on the extraction lever to deposit the amalgam compound.

WARNINGS:

Use only for the use described above.

The medical device has to be used by qualified medical-dental personnel.

The medical device is supplied in NON-STERILE condition therefore it must be carefully cleaned and

sterilized before use in autoclave up to 135 °C, it cannot be sterilized with dry air. Perform autoclaving at 121 °C for 20 minutes and 1 atm, 134 °C for 8 minutes and 2 atm. Avoid contact with metal items inside the autoclave.

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sterilized before use in autoclave up to 135 °C, it cannot be sterilized with dry air.

Perform autoclaving at 121 °C for 20 minutes and 1 atm, 134 °C for 8 minutes and 2 atm.

Avoid contact with metal items inside the autoclave.

Use the device up to a maximum of 100 sterilization cycles.

Before the sterilization, the tip must be unscrewed. To replace the tip, simply unscrew it. Resists all chemical agents. It is not attacked by amalgam.

After each use, the amalgam gun must be disassembled and cleaned from possible amalgam residues. These instructions are those of the correct medical-dental practice.

It resists at all disinfectant chemicals. Melts at 166 °C.

Store in a clean and dry place, away from direct heat, dust and atmospheric agents.

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Vertrieb durch Müller & Weyandt GmbH · Reichardsweide 40